

FRONT



SMART COOLING LID

QUICK GUIDE



FREEZE ICE PACK
6+ HOURS
BEFORE USE



BLUE = SAFE

46–64°F (8–18°C)

Ideal for insulin storage



ORANGE = TOO WARM

72°F+ (22°C+)

Move to a cooler place



1 Charge ≈ 600 Uses

Up to 2 Months Battery Life



ACTIVATE WARRANTY
GET +1 YEAR PROTECTION

Scan to register your order
Cooling tips & support included



BACK

QUICK TIPS



Avoid direct sunlight
Keep in a cool place.



Red warning icon flashing
= Temperature too high.

UNIT SWITCH



TAP THE ICE MOUNTAIN

3 times quickly

Switch between
°C ↔ °F

FAQ



Why is the orange light on?

Temperature is too high (72°F+). Move to a cooler place.



Why is the red warning icon flashing?

Temperature has exceeded the safe range.



Unable to switch temperature?

Tap the ice mountain 3 times quickly.



SETUP SUPPORT

Diabetes_Care@outlook.com